




S

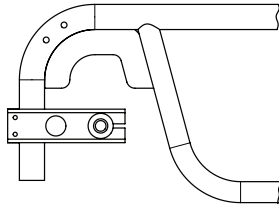


M

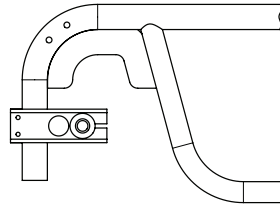

L

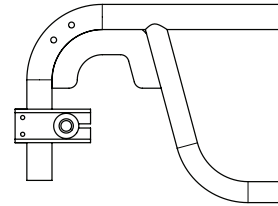

XL

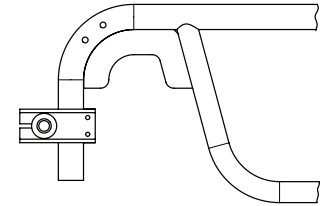

Grösse Size	
S	30.100.5380
M	30.100.5400
L/XL	30.100.5360

02.001.1710

10.020.2614

10.050.0046

10.016.2624

Standard

3°

6°

Grösse Size	
S	32.064.0130
M	32.064.0131
L/XL	32.064.0132

Sitzbreite Seat width	Standard	Radsturz 3° Camber 3°	Radsturz 6° Camber 6°
35	31.690.0100	31.700.0390	31.700.0460
38	31.700.0320	31.700.0400	31.700.0470
40	31.700.0330	31.700.0410	31.700.0480
42	31.690.0110	31.690.0130	31.690.0150
45	31.690.0120	31.700.0430	31.700.0500
48	31.700.0360	31.690.0140	31.690.0160

Erstellt Prepared	17.08.2007	TN	Benennung / Description Hinterradachse Rear wheel axle	Artikelnummer Art. no. 50.069.0600	Blatt / Anz. Sheet / no. of 1 / 1 Index / Rev.
Geändert Modified		-			
Gepr./Freig. Approved	30.04.2008	TN			
Status			Rollstuhl Wheelchair	LAE	
LEVO AG CH-5610 Wohlen www.levo.ch					